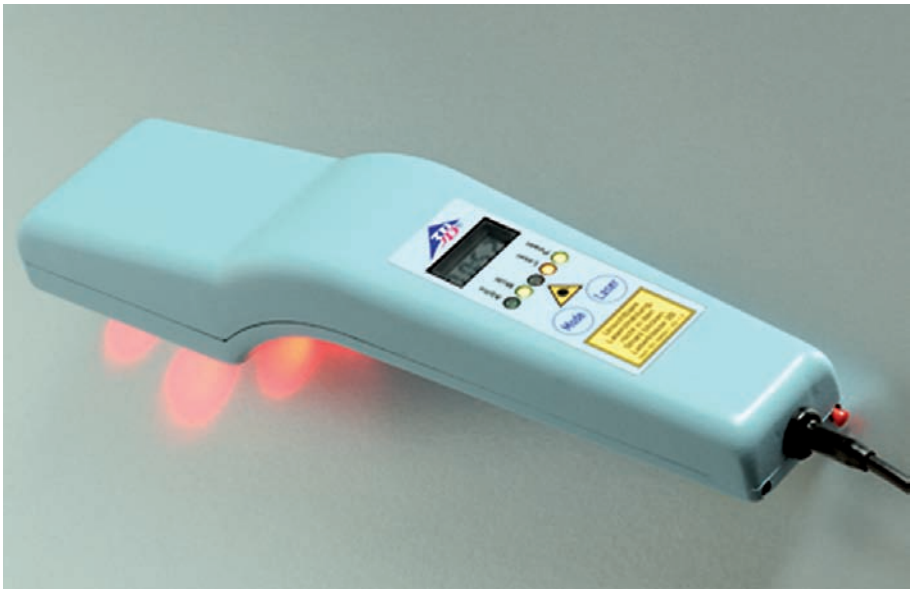
...going one step further


Laserdusche

Gebrauchsanweisung
Operating instructions
Mode emploi
Istruzioni per l'uso
Manual de instrucciones

www.3b-laser.de

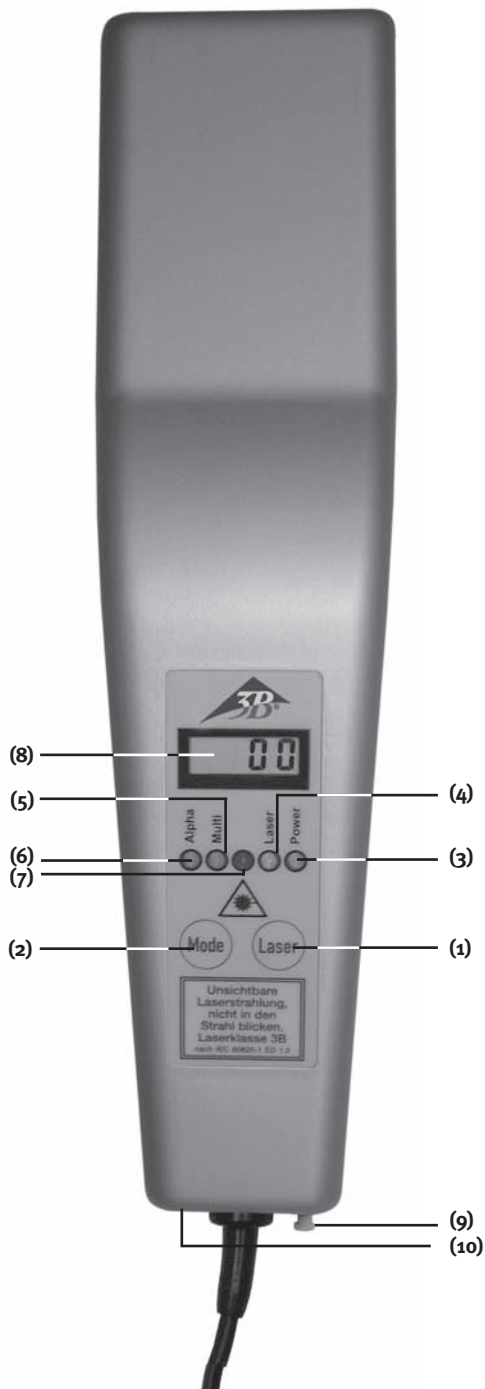LDU 8015 PN

Funktionstasten

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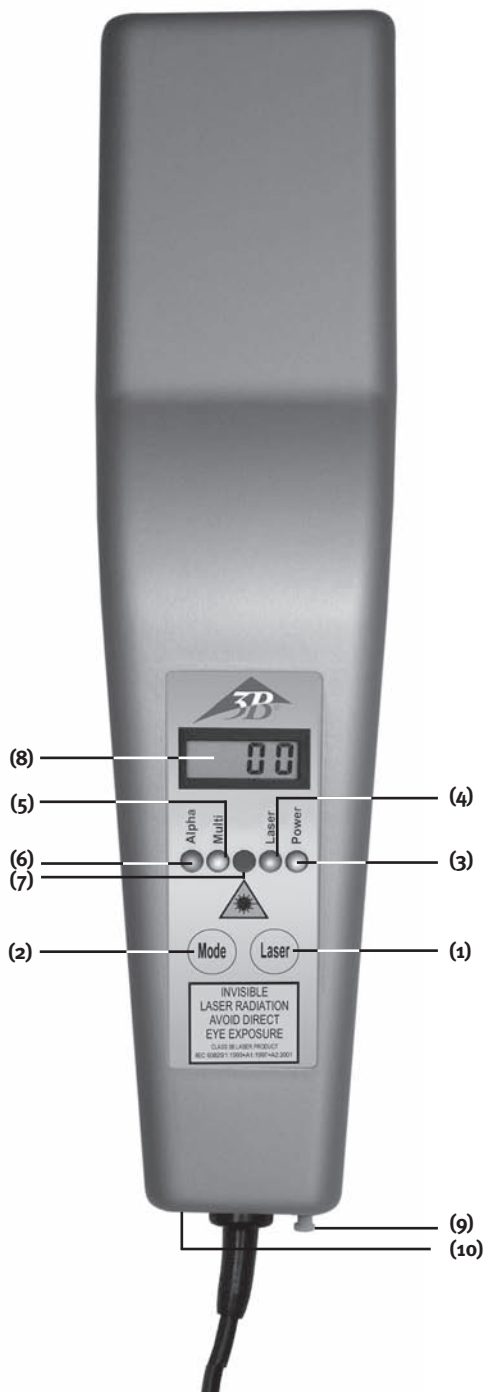
LDU 8015 PN

Function keys

- | | |
|-----------|--|
| (1) LASER | This key is used to activate the laser shower. The yellow “Laser” LED above the key lights up to indicate this, and the therapy time is running. |
| (2) Mode | This key is used to switch between continuous wave (CW), multi-frequency and alpha-frequency modes. The multi-frequency mode is preset. |

Display elements

- | | |
|-------------------|--|
| (3) Power | Indicates, that the laser shower is ready for operation. |
| (4) Laser | Indicates, that the laser shower is active. |
| (5) Multi | Indicates, that the laser shower is operating in multi-frequency mode. |
| (6) Alpha | Indicates, that the laser shower is operating in alpha frequency mode. |
| (7) Fault lamp | If the red lamp between the multi and laser lamps lights up permanently, contact the service department. |
| (8) Timer | Indicates the therapytime in min and sec. |
| (9) Red socket | For key pin, laser shower can only be used with the pin inserted. |
| (10) Black socket | For key pin, if you wish to lock the device. |

Starting up

Insert the key pin into the red socket to the right of the cable outlet of the laser shower. Connect plug-type power supply unit to earthed socket (100V-240V~). The green LED "Power" lights up when the power supply is applied. The value "oo" is output on the display.

The modulation mode "Multifrequency" is preset. The "Multi" LED is on as confirmation. We recommend, that you always work in the mode "Multi".

Using the "Mode" key you can change the modulation to Alphafrequency. For the mode "CW = continuous wave" you must press the Mode key for approx. 1 Sec. The LED's Multi and Alpha are off as a confirmation.

Activate the laser shower using the "Laser" key. The "Laser" LED lights up as confirmation, and the therapy time is incremented. In addition, 4 intensive red light diodes identify the treatment area. Press the "Laser" key again to stop the laser therapy and to set the timer to "oo" again.

The laser shower carries out a self-test. A fault in one or more laser diodes is indicated by a red lamp (between "Multi" and "Laser"). Contact the service department if this is the case.

The laser shower switches off automatically after 20 minutes.

Following treatment, switch off the laser shower using the "Laser" key, and remove the key pin. The unit is then no longer available for use. Store the key pin by inserting it into the black socket.

Applications

The MKW laser shower LDU 8015 PN can be used in the following fields: dermatology, orthopedics, sport physiotherapy. A further field of application is vitality medicine.

The effect of laser light can be to reduce pain, inflammation and swelling, and to heal tissue. With certain prerequisites, it stimulates faulty biological processes, a reason why lasers are also referred to as light vitamins. The objective of laser treatment is to accelerate natural healing processes. Laser treatment can often also restart disturbed healing processes. For healthy persons, the MKW laser shower is used to optimize vitality, to increase well-being, to strengthen resistance and as a measure towards preventing sicknesses.

Contraindications

The following contraindications are referred to in the laser literature. These also apply to the MKW laser shower LDU 8015 PN:

Absolute:

- ▣▣▣ Irradiation of eyes
- ▣▣▣ Photosensitivity (light dermatosis)
- ▣▣▣ Tumor patients
- ▣▣▣ Open fontanel, epiphyseal cartilages (long bones)
- ▣▣▣ Thyroid gland hyperactivity

Relative:

- ▣▣▣ Pacemaker patients (thorax)
- ▣▣▣ Epilepsy (head)
- ▣▣▣ Pregnancy (stomach, back)
- ▣▣▣ Endocrine organs (thorax gland, testicles, ovaries)

Therapyhindrances: deep x-ray, permanent medication, chemotherapy.

Use

The following must be observed if you wish to achieve optimum success with the therapy:

- ▣▣▣ To keep reflection losses as small as possible, skin must be grease-free.
- ▣▣▣ The distance between the laser shower and the skin is a few mm, or direct contact.
- ▣▣▣ Move the laser shower over the skin slowly and uniformly, and only activate it when it is in the position for treatment.
- ▣▣▣ Only apply ointments, lotions, creams and ampoules when the laser treatment has been finished.
- ▣▣▣ Clean the laser beam outlet with a cotton pad using 70% ethanol (med. alcohol) following each contact with skin and when changing the patient.

The principle of increasing application applies to all treatments. Initial treatments should not be longer than 2 minutes. The maximum treatment duration/patient is 20 minutes. The laser shower is switched off automatically after this period.

Approximate values for treatment times are 1-3 minutes/20 cm² for dermatological indications and 2-5 minutes/20 cm² for orthopedic indications. 20 cm² corresponds to the laser outlet area of the laser shower.

Maintenance and care

To guarantee safe and hygienic operation, observe the following:

- ▣ Clean the laser beam outlet using 70% ethanol (med. alcohol) following each contact with skin and when changing the patient.
- ▣ Do not expose the unit to direct sunshine.
- ▣ You can clean the housing using a mild cleaning agent. Disconnect the mains plug before cleaning.
- ▣ Repairs must only be carried out by the manufacturer or by authorized persons. The applicators are sealed. Unauthorized opening of the unit cancels the guarantee. Contact the service department if a repair is necessary.
- ▣ Only use the laser shower indoors and at room temperature. The supplied laser warning symbol must be attached to the door.

Safety information

Please observe the regulations specified in the User's Guide, and also the warning information present on the laser shower.

Protect the laser shower from unauthorized use by means of the key pin. Also observe the appropriate accident prevention regulations for laser radiation.

As a result of its wavelength, the laser radiation is invisible, or only weakly visible. Do not look into the laser outlet opening, and do not irradiate into the eyes of other persons. **Closed eyes must not be irradiated either.**

The safety distance when looking directly into the beam is approx. 50 cm. When used improperly, lasers can result in eye damage, especially at distances smaller than the safety distance.

Make sure that no mirrors or other reflecting surfaces are located within the safety distance from the laser shower (reflection of laser beam). You must always wear the supplied laser protection glasses.

Only switch on the laser shower when in the treatment position (direct contact with skin, or only a few mm away from the treatment area). When changing the treatment position, switch off the laser shower, and only switch it on again when in the new treatment position. The "Laser" lamp, the 4 intense red light diodes as well as the therapy time display indicate whether the unit is switched on. Switch off the laser shower following the treatment, and remove the key pin.

Magnetic and electric fields may influence the laser system functions. Therefore do not use the laser system in the vicinity of equipment which generate large electromagnetic fields, e.g. diathermy units or mobile telephones.

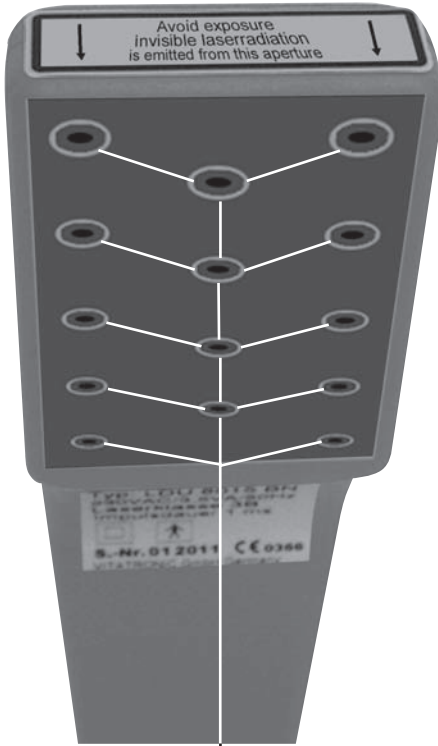
The device is subject to the WEEE regulations (waste of electrical and electronic equipment) 2002/96/EG and may not be disposed off as normal household waste. If you finally wish to terminate use of the laser system, please contact the technical service.



The manufacturer is only responsible for safety and reliability if the laser system is used in accordance with the User's Guide.


Warning labels

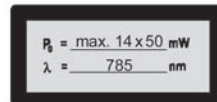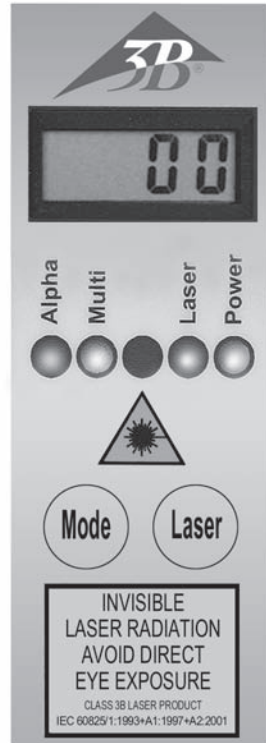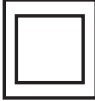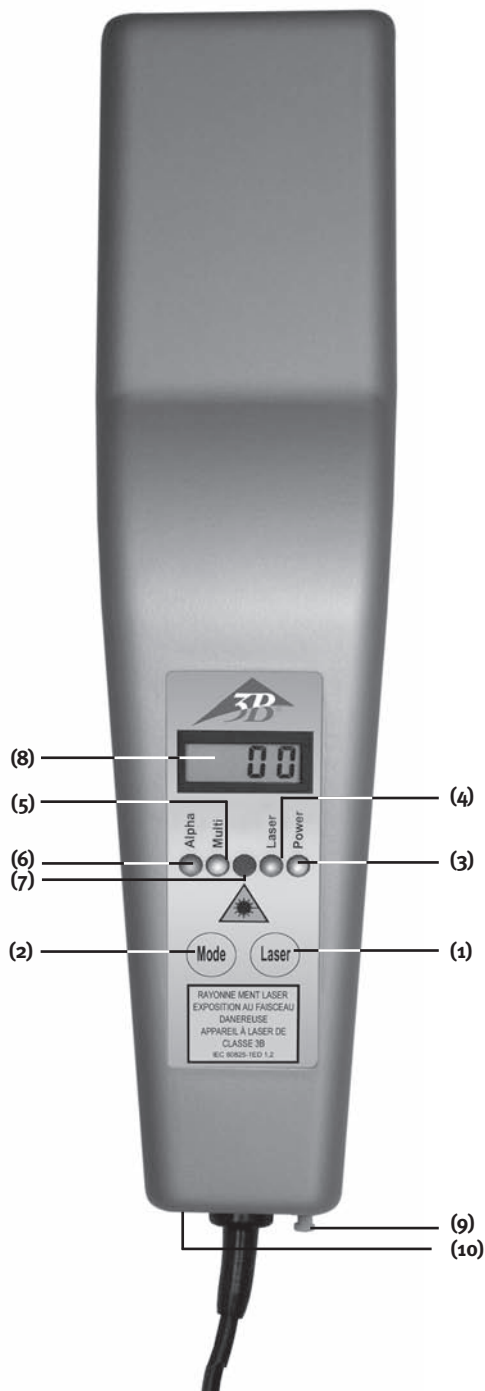
Outlet openings for laser radiation

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LIVETEC GmbH, Germany
Produttore Livetec GmbH, Marie-Curie-Strasse 8,
Distribuzione 3B Scientific GmbH
3B Scientific GmbH