PiKo[®] Monitors

Electronic Lung Health Monitor

*Redefining the way the world
detects and treats respiratory diseases*

User's Manual

English, German, Spanish,
French, Italian

Models

PiKo-1
PiKo-6

ATS/EU Scale Compliant

PiKo[®] *Monitors*

Table of Contents

<u>Language</u>	<u>Page</u>
English.....	1
Deutsch.....	11
Español.....	21
Français.....	31
Italiano.....	41

Intended Use

The PiKo-1 and PiKo-6 devices are designed to monitor respiratory conditions such as COPD and asthma.

The PiKo-1 measures the Peak Expiratory Flow (PEF) and the Forced Expiratory Volume in the first second of expiration (FEV₁). The PiKo-6 measures the Forced Expiratory Volume in the first second of expiration (FEV₁) and the Forced Expiratory Volume accumulated in the first six seconds of expiration (FEV₆).

The devices display the test results and also show the respective colour zone relative to the patient's reference values.

PiKo-1 - PEF or FEV₁

PiKo-6 - FEV₁, FEV₆ or FEV₁/FEV₆

In addition, the blow quality is evaluated to notify the patient to repeat the test in case of a cough or any suspect blow.

PEF - The fastest speed a person can blow air out of their lungs.

FEV₁ - The amount of air blown out in the first second of a forced exhalation.

FEV₆ - The amount of air blown out in the first six seconds of a forced exhalation (or the maximum amount if six seconds of exhalation was not achieved by the patient).

FEV₁/FEV₆ - The amount of air blown out in the first second of a forced exhalation, expressed as a ratio relative to the amount of air blown out in the first six seconds of a forced exhalation (or the maximum amount if six seconds of exhalation was not achieved by the patient).

Colour Zones - Display the test results in a “traffic light” colour format relative to the patient's reference value (PEF or FEV₁ for the PiKo-1, and FEV₁, FEV₆ or FEV₁/FEV₆ for the PiKo-6)

Q-Factor - Quality Factor notifies the patient to repeat the test in case of cough or any suspect blow.

Warnings!

NOTE: Please read all the information in this manual before using the PiKo-1 or PiKo-6. When the name PiKo is used, specific content applies to both PiKo-1 and PiKo-6. If the information only applies to one model of PiKo, it will be noted by its specific product name (PiKo-1 or PiKo-6).

The user should be under the care of a licensed healthcare professional. Healthcare professionals may recommend the PiKo as part of a disease management plan.

PiKo monitors are not designed or intended to be used as a primary diagnostic tool for asthma, COPD, or any other pulmonary disease.


Regardless of displayed test results, if signs or symptoms of chest tightness, shortness of breath, coughing or wheezing occur, contact your healthcare professional immediately.

Follow instructions carefully to get a correct reading.

If you use the PiKo above 1000ft elevation (300m), adjust your values by adding 1.5% of the measured value per 1000ft of additional altitude.


PiKo Description


1. Removable mouthpiece
2. Vent holes
3. Operate button
4. Colour zones
5. Zone indicator
6. Q-Factor indicator
7. Low battery indicator
8. IR communication window
9. PEF data, LPM (applies to PiKo-1 only)
10. PEF indicator (applies to PiKo-1 only)
11. Memory counter
12. FEV₁ (or FEV₆) indicator
13. FEV₁ (or FEB₆) data, litres
14. Battery cover, (behind)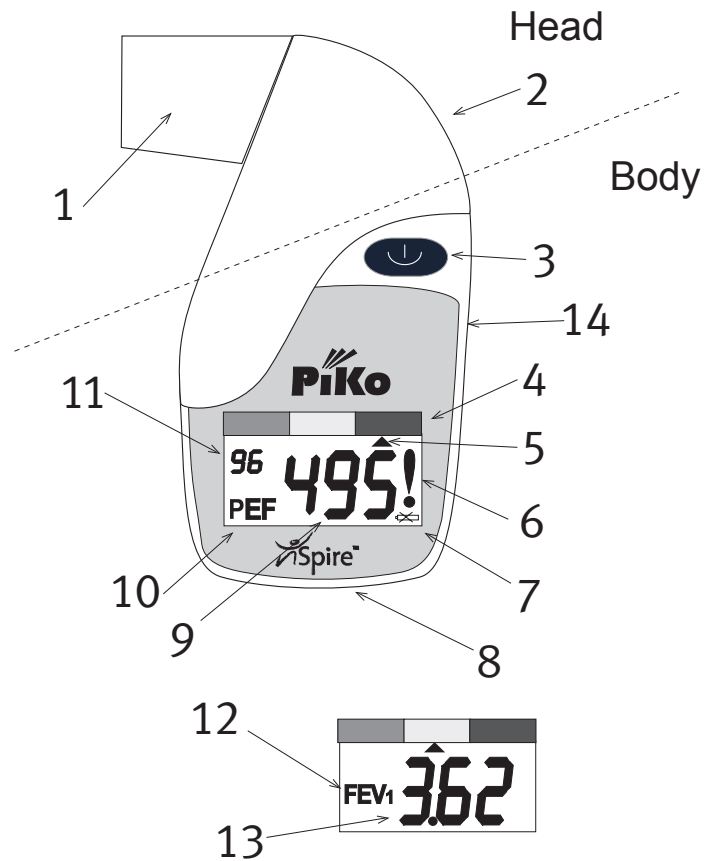


Using the PiKo

NOTE: If the LCD display is blank, press the Operate Button once and the PiKo display will show the last test results.

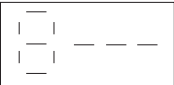1. Stand up. Hold the PiKo horizontally in your right hand with the display pointing up. DO NOT COVER THE VENT HOLES
2. Bring the PiKo up close to your mouth and press the Operate Button momentarily with your index finger. A short beep will be heard.
3. Inhale as much air as you can when a second beep is heard and the blow animation appears. 
4. Place the mouthpiece in your mouth and blow as hard as you can for at least 1.5 seconds (for the PiKo-1). In the case of the PiKo-6, continuously blow out for as long as possible, attempting to exceed a total of 6 seconds until an “end of blow” beep sounds. Keep the PiKo horizontal.

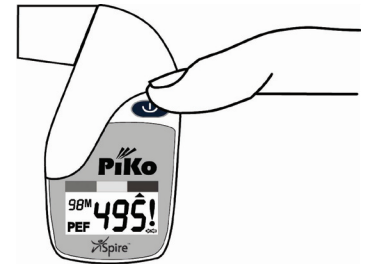
Your test results will be displayed and the Zone Indicator will show your colour zone if a reference value was set. If you wish to perform 2 or more tests, the PiKo will select and save the best results from all tests performed within 3 minutes of each other.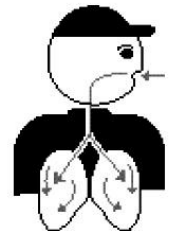

NOTE: Q-Factor Indicator (!) - a “!” symbol may show next to the reading, indicating one or more of the following occurred during the test:

- A cough was detected.
- The blow effort was not long enough. (FEV₁ - 1 second)
- The blow effort had a slow start.
- The result of the test was unnaturally low or high for your reference.

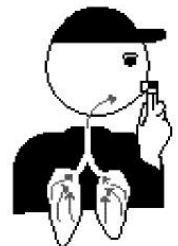When you have finished, always return the PiKo to its case. It will turn off automatically.

Setting Reference Values

One parameter (PEF or FEV₁ for the PiKo-1, and FEV₁, FEV₆ or FEV₁/FEV₆ for the PiKo-6) may have been selected at the factory to have a reference value assigned by the user in conjunction with their physician. Unless otherwise specified, the PiKo-1 is set up with the PEF as the reference type and the PiKo-6 is set up with the FEV₁/FEV₆ ratio as the reference type. After the reference value has been assigned, the selected parameter will then be compared with its reference and the result indicated by an arrow pointing to the appropriate colour on the colour zone. The zone indication may be turned off by setting the reference value to '0'.

1. Press the operate button quickly - 4 times within 2 seconds. The factory set reference type indicator will start blinking (FEV₁, FEV₆ or the FEV₁/FEV₆). The display will also show the stored reference value.
2. Release the button. Each additional press will advance the reference value display by 0.10 L for FEV₁ or FEV₆, or 0.01% for FEV₁/FEV₆.
3. Hold the button for 5 seconds to save the desired reference value. The appropriate indicator will stop blinking and a beep will sound.
4. The display will briefly show the new reference value and then the last test results. The Colour Zone in the memory review mode will reflect the new reference value.

NOTE: During the reference setting process, if the Operate Button is not pressed within 5 seconds, the PiKo will exit the setting mode without modifying the old reference.

The following colour zone parameters are used:

GREEN, YELLOW, RED

Green = test results over 80% of the reference value

Yellow = test results between 50 - 80% of the reference value

Red = test results under 50% of the reference value

Memory Review

The PiKo stores the last 96 test values and other data associated with the tests in its memory. If the memory becomes full with 96 tests and another test is done, the oldest test will be eliminated from memory. The newest test will then be number 96.

To scroll through the memory:

1. Press and hold the operate button for 5 seconds. The display will show the last test results containing the PEF (PiKo-1 only), FEV₁, FEV₆ (PiKo-6 only), FEV₁/FEV₆ ratio (PiKo-6 only), colour zone and the Q factor, if any. It will also show the respective memory counter location.
2. Release the button. Each additional press of the button will now display the next oldest stored test.

To exit the Memory Review Mode either press and hold the operate button for 5 seconds or avoid pressing the operate button for 20 seconds. The PiKo will then display the latest test result and will be ready for a new test.

All the test data stored in the PiKo memory can be downloaded to a PC using the optional PiKoNET Software (interface cradle included).

This software can be purchased on the **www.nspirehealth.com** website.

NOTE: For the PiKo-1 the entire memory can be erased by setting the PEF reference to 0.10, or the FEV₁ reference to 0.10. For the PiKo-6, the entire memory can be erased by setting the FEV₁ or the FEV₆ reference value to 0.10 or the FEV₁/FEV₆ ratio to 0.01.

Meaning of Sounds

Press Operate Button:	Short beep
Error during initialization:	Long beep
Blow command:	Short beep
End blow acknowledge:	Short beep
Acknowledge reference setting:	Short beep
Result within green zone (PiKo-1):	No beep
Result within green zone (PiKo-6):	Short beep
Result within yellow zone:	2 short beeps
Result within red zone:	Long beep
Any result containing (!):	Long beep

Meaning of Symbols

This symbol indicates that the user must read and understand all instructions and warnings.

This symbol indicates that this Class I equipment with measurement function complies with the European Union Medical Device Directive.

This symbol indicates that this device provides a certain level of protection against electrical shock.

This symbol indicates that this device is splash proof.

Maintenance

NOTE: This product is NOT user serviceable.

Cleaning the mouthpiece:

1. Detach the mouthpiece by ‘snapping’ it off.
2. Wash the mouthpiece with washing up liquid (a mild detergent) in cold water.
3. Rinse and dry thoroughly. DO NOT use heat to dry.


Cleaning the body of the PiKo:

1. Rinse with low flow/pressure water at room temperature. The top part of the unit may be cleaned with a solution of 5% washing up liquid (a mild detergent) in cold water, followed by rinsing.
2. Hold the PiKo upside down and shake to remove the water.
3. Wipe exterior with a clean, dry towel.
4. Allow to dry naturally and completely before further usage or storage.
5. To avoid damage to the PiKo: DO NOT immerse in water or use a dishwasher. DO NOT use a high-pressure water flow. DO NOT use any sort of active solvent for cleaning. DO NOT dry by heat, hair dryer, or a dishwasher. DO NOT insert anything into the vent holes or the mouthpiece opening.

Battery Replacement and Insertion:

PiKo units are supplied with batteries but they are not inserted. Please insert them per the following instructions:

1. Replace batteries when “Low Battery” indicator appears.
2. Open the battery door using a small coin. Insert the coin in the slot and turn counter clockwise. Line up the coin slot with the arrow on the body of the PiKo. DO NOT try to turn past the arrow.
3. Remove plastic spacer and discard
4. Use 2 type 357, silver oxide button cell batteries or equivalent.
5. Replace the battery cover by lining up the coin slot with the arrow on the body of the PiKo and turning to the right to the other indicator on the body of the PiKo. DO NOT turn past the lock (-) indicator.

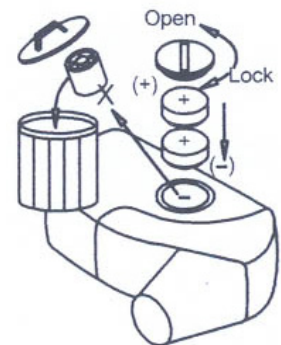Technical Specifications

PEF:	Range 15 - 999 LPM, (1 LPM resolution)
FEV ₁ :	Range 0.15 - 9.99 litre, (0.01 litre resolution)
FEV ₆ :	Range 0.15 - 9.99 litre, (0.01 litre resolution)
FEV ₁ /FEV ₆ :	Range 0.01 - 0.99, (0.01 resolution)
Accuracy:	PEF: 5% or +/- 20 LPM, (whichever is greater) FEV ₁ : +/- 3.5% or 0.1 litre, (whichever is greater) FEV ₆ : 4% or 0.1 litre (whichever is greater)
Sensor:	Pressure/flow sensor technology (patented)
Memory:	96 test scores containing: PEF, FEV ₁ , FEV ₆ , FEV ₁ /FEV ₆ , Colour Zone, Q-Factor
Colour zones:	3 colour zones (Green, Yellow, Red)
Reference values:	Set by user for PEF (for PiKo-1) and FEV/FEV ₆ (for PiKo-6); Default settings: PiKo-1 = PEF 500 LPM, PiKo-6 = FEV ₁ /FEV ₆ 0 (disabled)
Quality factor:	Warning and indicator for cough or abnormal blow
Sounds:	4 patterns for different indications and warnings
Communication:	Bi-Directional IR port
External settings:	Possible settings using the optional PiKoNET Professional software: <ul style="list-style-type: none">- Colour zone limits (in 10% increments)- Use of FEV₁ for PiKo-1 and FEV₁ or FEV₆ for PiKo-6 as an alternate reference type
Battery:	2x type 357 silver oxide button cells (or equivalent)
Battery life:	PiKo-1: approx. 1 year; PiKo-6: approx. 6 months (based on average of 6 blows/day)
Dimensions:	75 X 35 X 20 mm, (3 X 1.3 X 0.8 inches)
Weight:	35 grams (1.25 oz)
Accessories:	Plastic Mouthpieces - part number:393535
Resistance:	<2.5 cm H ₂ O/l/s @ 14 LPS or lower
Operating temp.:	10 to 38°C (50 to 100 °F)
Storage temp.:	(-20) to 60 °C (-4) to 140°F)
Humidity:	0 - 100% relative humidity
Barometric:	550 to 780 mmHg
Performance:	ATS1994 (monitoring), AS/NZS-4237: 1994, EN13826: 2003
Safety:	EN60601-1, EN60601-1-1, EN60601-1-2, EN13826:2003, Type BF patient applied part, and IPX ₄
Regulatory:	PiKo-6 FDA 510(k); PiKo-1 FDA 510 (k) for OTC; CE (0086), Class I with measurement function
Contracts:	GSA V797P-4063a; UK NHS Drug Tariff prescribable on FP10
USA Patent:	US # 6,447,459 B1

Measurement Scale

For many years there have been two standards for peak flow measurement - the Wright Standard, and the American Thoracic Society's standard (ATS). In 2004 a new European Union Standard was adopted, known as the EU Standard, EN13826: 2003. The PiKo-1 complies with both the ATS and the new EU Standards. The PiKo-1 measures PEF in litres per minute BTPS (Body Temperature and Pressure Saturated). There is a difference between the earlier Wright Scale devices and this PiKo-1, which complies with the ATS and the EU scales. Do not directly compare readings of the PiKo-1 with readings from a unit using the Wright Scale.

Warranty

The PiKo is warranted for 6 months against faulty parts or workmanship. Batteries are not covered by warranty. Failure to use in accordance with the instructions could render the warranty invalid. This does not affect statutory rights.

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U.S.A. Patent # 6,447,459 B1
nSpire Health, Inc.
nSpire Health Ltd.
nSpire Health GmbH